



Companion Animal Implant Procedure

Identipet

Precautions: Single-use material. Do not reuse the applicator. Destroy the applicator according to local regulations after use. If not used when sterile packaging is opened, do not autoclave, as heat may damage internal circuitry. Alternatively, cold sterilization of the complete applicator system may be accomplished by soaking the applicator with preloaded transponder for 15 minutes in chlorhexidine solution (3 oz/gal).

Implant Site: To comply with international standards the implant site for dogs and cats is a subcutaneous injection between the shoulder blades. All Vets, SPCAs and Welfare Societies are requested to adhere to this. Scanning however should still be done on the left hand side of the neck as well as between the shoulder blades.

Implant Procedure:



Check the integrity of the sterile pack.
Verify transponder operation with Pocket Reader.

Please be sure to draw the attention of the pet owner to our Annual Service Fee of R25-00, which is set out in detail on the reverse of the registration form.



Scan the pet for the presence of a microchip.

Apply an antiseptic anti-microbial preparation to the implant site.

Using your thumb and index finger, pull some loose skin up at the injection site while pinching it lightly. Roll your index finger over your thumb to release some skin.

Position the implant needle 45° to the skin surface.

Pierce the skin.

Once the skin is penetrated, position the implanter parallel to the skin and penetrate subcutaneous tissue until the needle is inserted to the hub.

Push the thumb button forward until fully extended.

Withdraw the needle slowly.

Ensure that the transponder has been injected, and is not withdrawn with the needle.

Rub the the implant site with the thumb and hold for a few seconds.



Scan to confirm the implant.



Complete the Registration Form in full, ensuring that the information is correct and legible (see example on page two).

IDENTIPET™ REGISTRATION FORM

NB

SUCCESSFUL RECOVERIES OF ANIMALS DEPEND ON ACCURATE AND COMPLETE INFORMATION

405C237B70**EXAMPLE**

PLEASE COMPLETE IN BLOCK CAPITALS OWNER INFORMATION

Mr Ms Miss Dr **D A SMITH**

I.D. Number **1 9 5 9 1 0 1 2 0 1 4 5 0 3 8**

Postal Address **P O BOX 23 SANDTON** Physical Address **138 GIBSON ST SANDOWN**

Postal Code **1 9 4 5**

Home Telephone No. **0 1 1 - 7 8 4 8 4 7 2** Cell Number **0 8 3 - 7 8 4 8 4 7 2**

Work Telephone No. **0 1 1 - 4 4 7 6 2 3 3** Cell Number **0 8 3 - 6 1 3 3 5 2 3**

Fax No. **0 1 1 - 7 8 4 8 4 7 2** E-mail **smith@home.co.za**

ANIMAL INFORMATION

Type of Animal Dog Cat Horse Bird Reptile Game Other Specify

Name of Animal **SPARKY** Breed **JACK RUSSEL**

Description **BROWN AND WHITE** Sex M F Sterilized Y N

Species **CANINE**

Date of Birth **2004** Name of Breed Society **KUSA**

VETERINARIAN INFORMATION

Name of Implanting Veterinarian **DR W DANIEL** Date of Implant **24/11/2004**

I certify that I have today implanted the above described animal in the prescribed implant site with the implant number as recorded above.

13784 Vet's P.I.N. Number

White Copy - IDENTIPET • Pink Copy - VETERINARIAN • Yellow Copy - BREED SOCIETY • Green Copy - OWNER

White Copy - IDENTIPET • Pink Copy - VETERINARIAN • Yellow Copy - BREED SOCIETY • Green Copy - OWNER

White Copy - IDENTIPET • Pink Copy - VETERINARIAN • Yellow Copy - BREED SOCIETY • Green Copy - OWNER

I accept the conditions of Sale / Service

White Copy - IDENTIPET • Pink Copy - VETERINARIAN • Yellow Copy - BREED SOCIETY • Green Copy - OWNER

Post white copy to: Identipet, P.O. Box 215, Lanseria, 1748 • Tel: (011) 957-3456 • Fax: (011) 957-2784 • Web: www.identipet.com