




# Equine Implant Procedure

Identipet

## Implant Site:

The microchip is implanted on the left hand side of the horse's neck, within the nuchal ligament in its middle third or at the halfway point between the ears and the withers.

## Implant Procedure:

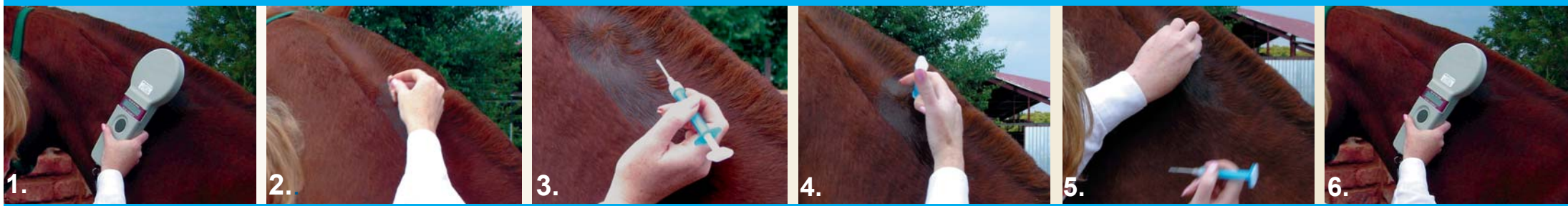

Check the integrity of the sterile pack.

Verify transponder operation with Pocket Reader (see Operator's Manual) prior to insertion.

1. Scan the horse's neck to ensure that there isn't already a microchip implanted. *(All Thoroughbred horses born after 1999 should already have an Identipet microchip. See note.)*
2. Apply an antiseptic anti-microbial preparation.
3. Position the implant needle 45° to the skin surface. Pierce the skin.  
Once the skin is penetrated, penetrate tissue until the needle is inserted to the hub.
4. Push the thumb button forward until fully extended.  
Withdraw the needle slowly.  
Ensure that the transponder has been injected, and is not withdrawn with the needle.
5. Apply pressure to the implant site and hold for a few seconds.
6. Scan the horse to confirm the implant.

Complete the Registration Form in full after implanting the microchip, ensuring that the information is correct and legible (see example).

**Note:** The Jockey Club of South Africa made racehorse identification with Identipet™ microchips mandatory from January 1998. All Thoroughbred horses born after August 1999 should have an Identipet™ microchip and the relevant information recorded on the Jockey Club's database. If the horse has an Identipet™ microchip, registration onto the Identipet™ database is free of charge. Keep a record of the microchip number, ask the owner to contact us and we will send a registration form to them to be completed.

# IDENTIPET™ REGISTRATION FORM

**NB**

SUCCESSFUL RECOVERIES OF ANIMALS DEPEND ON ACCURATE AND COMPLETE INFORMATION

**EXAMPLE**

405C237B70

## PLEASE COMPLETE IN BLOCK CAPITALS OWNER INFORMATION

Mr  Ms  Miss  Dr  **D A SMITH**

I.D. Number **1 9 5 9 1 0 1 2 0 1 4 5 0 3 8**

|                |                   |                  |                      |
|----------------|-------------------|------------------|----------------------|
| Postal Address | <b>P O BOX 23</b> | Physical Address | <b>138 GIBSON ST</b> |
|                | <b>SANDTON</b>    |                  | <b>SANDOWN</b>       |
| Postal Code    | <b>1 9 4 5</b>    |                  |                      |

Home Telephone No. **0 1 1 - 7 8 4 8 4 7 2** Cell Number **0 8 3 - 7 8 4 8 4 7 2**

Work Telephone No. **0 1 1 - 4 4 7 6 2 3 3** Cell Number **0 8 3 - 6 1 3 3 5 2 3**

Fax No. **0 1 1 - 7 8 4 8 4 7 2** E-mail **smith@home.co.za**

## ANIMAL INFORMATION

Type of Animal Dog  Cat   Horse  Bird  Reptile  Game  Other Specify

Name of Animal **BENEDICT** Breed **WELSH SECTION B**

Description **GREY 13.2hh** Sex  M  F Sterilized  Y  N

Species **EQUINE**

Date of Birth **1998** Name of Breed Society **WELSH PONY & COB SOCIETY**

## VETERINARIAN INFORMATION

Name of Implanting Veterinarian **DR W DANIEL** Date of Implant **24/11/2004**

I certify that I have today implanted the above described animal in the prescribed implant site with the implant number as recorded above.

- This implant carries a lifetime guarantee.
- Computer registration is included free of charge.
- Annual Service fee is payable to IDENTIPET (Pty) Ltd.
- See reverse for conditions of sale.

I accept the conditions of Sale / Service

Veterinarian's Signature

**13784**

Vet's P.I.N. Number

Owner's Signature

White Copy - IDENTIPET • Pink Copy - VETERINARIAN • Yellow Copy - BREED SOCIETY • Green Copy - OWNER

Post white copy to: Identipet, P.O. Box 215, Lanseria, 1748 • Tel: (011) 957-3456 • Fax: (011) 957-2784 • Web: www.identipet.com